

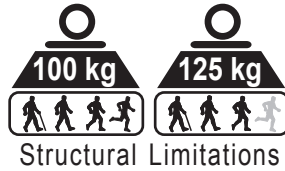


Multiflex Ankle Pylon - 30mm dia 125kg


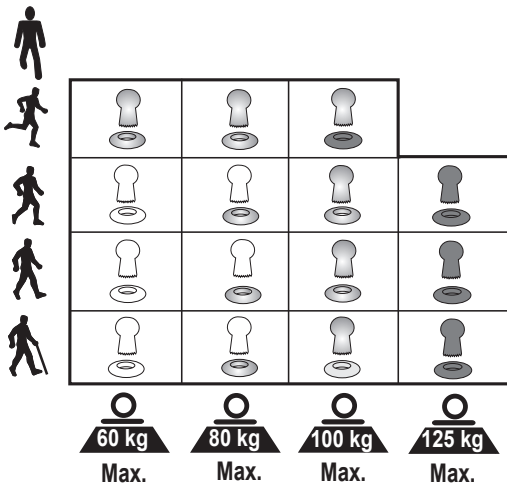


409241 - 243 30mm





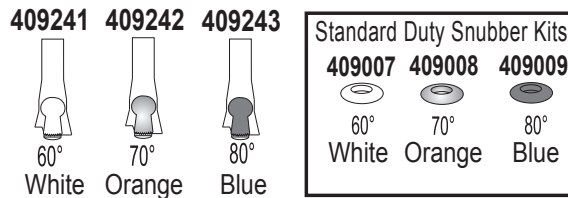


Notes:

Excessive dorsiflexion at ankle fit a harder snubber and vice versa.
Excessive plantarflexion at ankle fit a harder ball and vice versa.
Trans-femoral amputees may require a softer plantarflexion action than Trans-tibial.
Double amputees may require stiffer snubbers than single amputees.

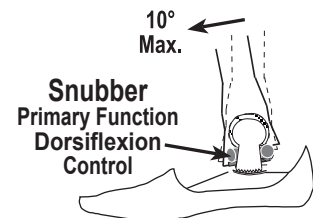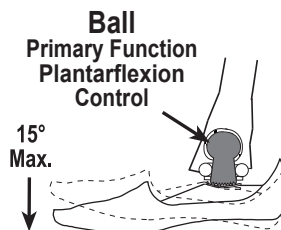

This information is intended for prescription guidance only. Structural limitations are shown above.

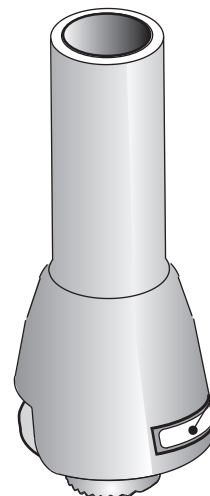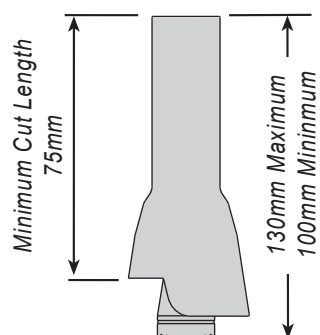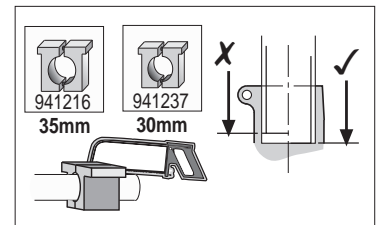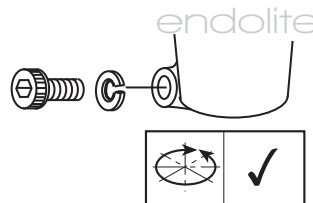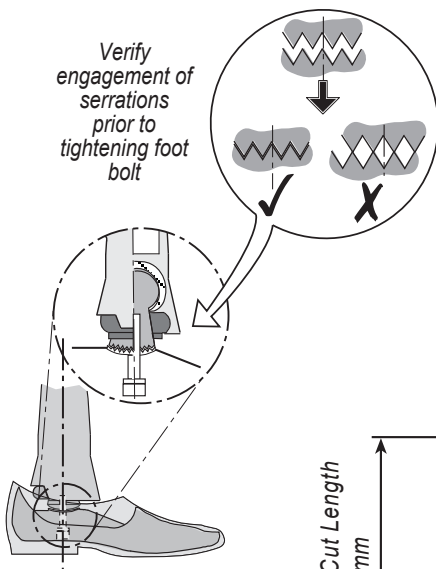

Mark label to show grade of ball assembly and snubber fitted. If grade of either is changed re-label (400014) accordingly